

WHEELCHAIR ELECTRIC HANDCYCLE



USER'S GUIDE

CNEBIKES CO.,LTD

Technical parameters

Frame material

Steel

Motor

36v 500w in wheel

Battery

36v 8ah 10.4ah 11.6ah 14ah

Controller

Intelligent controller

Charger

AC 100v-240v,2amps smart charger

Tyre

INNOVA 12 1/2×2.4

Speed limit

25km/h

Brakes

Disc brakes and hand brake

Light

Front led light with horn

Display

LCD5 display

Throttle

Half-twist throttle

Motor

36v 500w in wheel

Top speed is limited to 25km/h

Disc brakes & hand brake raking force




Battery

36v Polly case battery



Safety switch

Need to screw down the safety switch for the handcycle can operate.

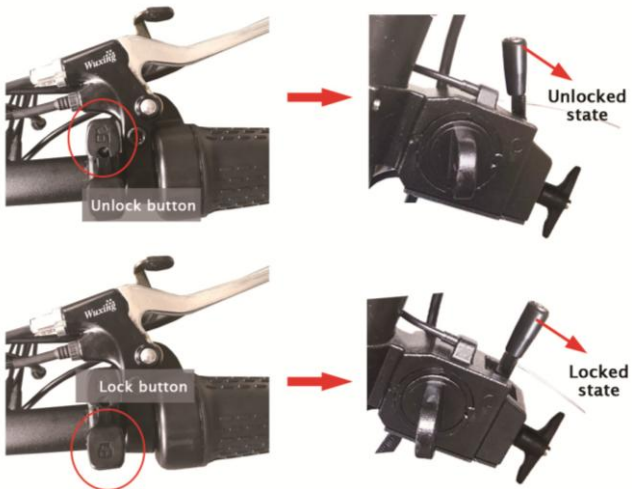

Hand brake

When you parked on a slope, the hand brake lock will stop wheelchair from reversing.



LCD5 display,level 1-3 is advanced,level
-1 is reverse.



Arm of wing

Reasonable shape and length can provide
plenty of legroom



Instruction

Sit in wheelchair & approach attachment



Attach wing couplings to wheelchair

(*The most important is that the red screw can be Adjust to fit wheelchairs with different tube size.)



Push handlebar forward,until wheelchairs front wheels are off the round



Wind in safety switch lever



Press on/off button&go



Mode setting

DST:Distance

ODO:Mileage

VOL:Battery voltage

LCD Display operation and setting

1、ON/OFF

Hold  button long to turn on the power, and hold  button long for a second time to turn off the power. when the motor stopped driving and when the e-bike is not used for a consecutive 5 minutes, it will automatically shut down and turn off the motor power supply.

2、Gear switch



Hold  or  button shortly to switch 1-3 file gear. 1 is for the minimum power, gear 3 is for the highest power. gear -1 is with reverse function

3、6km/h assist promotion function



Hold  button  and  flashes, the vehicle drives at the speed not more than 6km/h. release button, the function is invalid.

4、Cruise function



After the cruise is turned on, the trip riding speed is greater than 7km/h, hold  button long and enter cruise, the CRUISE lit. brake or hold any button to cancel.

5、Display and delete of single data



After power on for 5 second,
hold ▲ and ▼ button at the
same time,single trip riding
time (TM) and single trip
distance (DST) flash,hold 🔌
button shortly,the content of

both is cleared.if failed holding the button within 5
seconds,it will automatically return the display
interface after 5 seconds,original content is
preserved.

6、Hold 🔌 button to turn off the display and the
power supply of controller.

7、Set maximum riding speed



After power on for 5 seconds,
hold ▲ and ▼ button at the
same time Maximum riding
speed km/h and MXS flash,
hold ▲ or ▼ button shortly
to set The maximum riding

speed (default 25km/h).hold 🔌 button shortly and
go to the next parameter settings.

8、Wheel diameter setting



The wheel diameter will be set after finishing setting the maximum riding Speed, wheel diameter specifications

flashes hold  or  button

shortly to set The specifications of wheel diameter. select the rang 6,8,10,12,14,16,18, 22,24,26,700c and 28 inches.hold  button shortly and go to the next parameter settings.

9、Set the metric units



The metric units will be set after finishing setting wheel diameter, km/h and km flash.

hold  or  button shortly and select metric units of

speed and Mileage in synchronization.

Display	Metric	Imperial
Riding speed	Km/h	MPH
Total distance	Km	Mil

10、KM/H and Km stop flash after metric units setting is completed. Hold  button shortly again to re-enter the maximum riding speed setting interface; or hold  button long to exit from setting environment of routine projects and save the setting values, returning to display 1.

Warning!!!

When do emergency braking, please do not only brake the hand cycle, **but also you must brake your rear wheel of wheelchair itself**

Normally, please do not brake hardly, try to brake it slowly!!!

The max speed is 25km/h by default. if you want to be faster, you can set on the display.

We suggest you not make it too fast!!!

Enjoy your riding!

CNEBIKES CO.,LTD

Website:www.cnebikes.com

E-mail:info@cnebikes.com

Tel: 0086-51985809291 (8005)

Address:No3.Renmin West Road.Wujin

District.Changzhou City.JiangSu.China